

BIONIX OTOCLEAR® EAR IRRIGATION



OtoClear® Spray Wash Kit:

- Manual, intermittent lavage.
- Delivers greater volume compared to syringes or other manual devices.
- Efficient design allows for two-handed procedure.

Includes:


- 1 Spray Wash Bottle
- 20 OtoClear® Tips
- 1 Ear Basin




Usage Instructions



Insert *OtoClear* tip fully into the ear canal. The flared design of the *OtoClear* tip protects against over-insertion.

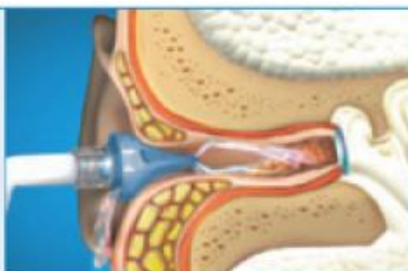

Begin irrigation. Note: The *OtoClear* tip directs the water towards the walls of the ear canal creating a turbulent lavage that effectively dislodges the cerumen. The exit portals control back-splash and deliver the effluent into the basin.

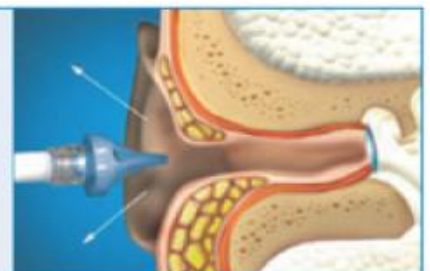

When procedure is complete, remove and dispose of the single-use *OtoClear* tip.



TEL: 0 100 350 238

FAX: 086 477 9020

E-MAIL: info@ububelemedical.co.za

BIONIX OTOCLEAR® EAR IRRIGATION



Bionix OtoClear® Ear Irrigation helps healthcare workers provide safe and comfortable ear irrigation without the backslash, at half the cost of a self-constructed device.

► PROBLEM:

Traditional ear irrigation shoots water directly into the ear canal, which can be painful, dangerous and messy.

► SOLUTION:

OtoClear® Ear Irrigation Tips, with Gentle Touch Design, directs water to the ear canal wall, with three divergent streams, instead of directly at the Tympanic Membrane. And the flared design prevents backslash.



► RESULTS:

A safe, clean ear irrigation tip at half the cost of a self-constructed device.

Ordering Information

- **UB-7290** **Bionix OtoClear® Spray Wash Kit**
- **UB-7200** **40 single-use tips per box**
- **UB-3700** **Ear Irrigation Basins (box of 3)**



TEL: 0 100 350 238

FAX : 086 477 9020

E-MAIL: info@ububelemedical.co.za